RosetteSep™ DM-M Density Medium

REF 15725

100 mL

ENGLISH

INTENDED USE

RosetteSep™ DM-M Density Medium is a density separation medium designed specifically for use as part of the RosetteSep™ kit for the enrichment of human myeloid (CD33⁺) cells from human whole blood (Catalog #15272HLA).

PRODUCT SPECIFICATIONS

RosetteSep™ DM-M Density Medium

REF #15725

Density: 1.085 g/mL

QUALITY CONTROL

RosetteSep™ DM-M Density Medium is manufactured using aseptic technique and tightly controlled processes. Each lot of RosetteSep™ DM-M Density Medium is sterility tested according to USP methods.

STORAGE AND STABILITY

Store at 15 - 25°C. Storage at 2 - 8°C is acceptable, but ensure that the medium equilibrates to 15 - 25°C and invert bottle to mix contents before use. Keep protected from direct light. Product stable at 15 - 25°C until expiry date as indicated on label.

WARNINGS AND PRECAUTIONS

1. For professional users only.
2. This product is for in vitro diagnostic use.
3. Do not use bottle if contents have leaked. Unused density medium may be disposed of according to standard laboratory procedures for non-hazardous liquids.
4. This product should be handled by trained personnel observing good laboratory practices. Once this product is added to human cells, treat the suspension as potentially biohazardous. Handling of reagents and disposal of wastes should observe all local, state, or national regulations.
5. This product is a potential irritant to eyes, respiratory system, and skin. This product may also be harmful if ingested. Avoid exposure through skin, eye contact, inhalation, and ingestion.

HANDLING AND DIRECTIONS FOR USE

Please refer to the RosetteSep™ Product Information Sheet for RosetteSep™ HLA Myeloid Cell Enrichment Kit, Catalog #15272HLA, for detailed instructions.

Note: Should not be used with any other RosetteSep™ enrichment kits. The density of RosetteSep™ DM-M Density Medium is not optimized for the isolation of other cell types.

TECHNICAL ASSISTANCE

For technical support please contact us by email at techsupport@stemcell.com or call either **+1.604.877.0713** or the European Toll-Free number 00800 7836 2355. For more information please visit www.stemcell.com.

If you require a printed copy or a translated version of this document in a certain language please contact technical support.

EC REP MDSS GmbH

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30175 Hannover, Germany

REF Catalog or reference number	LOT Batch code	 Use by: YYYY-MM
 Caution, consult accompanying documents	IVD In Vitro Diagnostic Medical Device	 For storage within temperature limits
 CE Mark	 Manufacturers identification (name & address)	EC REP Authorized EC representative in the European Community

STEMCELL Technologies Inc | 1618 Station Street, Vancouver, BC | V6A 1B6 | Canada | www.stemcell.com

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